PowerCel™ Adapter CI-7402

Instructions for use
English, Français, Español, Deutsch, Nederlands, Italiano

Labeling

The symbols below are used on the labeling for the product. Their meanings are as follows:

 0123	European Community Mark of Conformity. Authorized to affix the CE Mark in 2009		
REF	Model Number		Date of manufacture
LOT	Lot Number		Manufacturer
	Fragile		Do not get wet
	See Instructions for Use		
	Store at temperatures between -20°C (-4°F) and +55°C (131°F)		
	Suitable for atmospheric range between 70kPa and 106kPa, which is equivalent to 3000m above sea level to 380m below sea level.		

	Suitable for exposure to relative humidity between 0-95%
	Dispose of in accordance with applicable national and local regulations
EC REP	European authorized representative
UDI	Unique Device Identifier
MD	Medical Device

Warnings and Cautions

1. Small Parts Hazard: The PowerCel™ Slim, PowerCel™ Plus, PowerCel™ Adapter, BTE Power Adapter, Power Adapter Cable, Huggie and Filler Plug may cause choking if swallowed.
2. Do not use leaking batteries, as they may burn the skin.
3. Do not bite or chew the PowerCel Adapter or related products.
4. Do not expose products to liquid.
5. Do not use the PowerCel Adapter if the housing is cracked.
6. Do not loop cables around the neck, due to risk of strangulation.

7. Do not use broken, frayed or cracked cables.
8. To avoid an increased risk of strangulation, do not connect the Power Adapter Cables in any combination which would result in a total length (end-to-end) longer than 32 inches (81 cm).
9. While no injury cases have resulted, components of the system may become hot during normal use or a fault condition. If a particular device's temperature results in discomfort or pain, promptly remove the product and contact Advanced Bionics.
10. To prevent overheating resulting in possible harm to the user, do not cover or wear the PowerCel Adapter and battery under clothing next to the skin.
11. Do not attempt to open or repair the PowerCel Adapter. Do not continue to use if any part of it is damaged. Unauthorized opening will void the warranty and may compromise system performance.

Intended Use

The PowerCel Adapter is intended to be used to power the Auria™ or Auria™ Harmony™ Sound Processor with a BTE Power Adapter, a Power Adapter Cable, and either a PowerCel Slim or a PowerCel Plus.

Package Contents and Related Products

Package Contents:

One PowerCel Adapter (CI-7402)*

Related products are:

PowerCel Slim (CI-5520)*, PowerCel Plus (CI-5540)*

Power Adapter Cable (CI-7405)*

BTE Power Adapter (CI-7404)*

Auria Sound Processor (CI-5230)*

Auria Harmony Sound Processor (CI-5235)*

Bilateral Power Adapter (CI-7406)*

See Figure 1.

*Applied part per IEC 60601-1

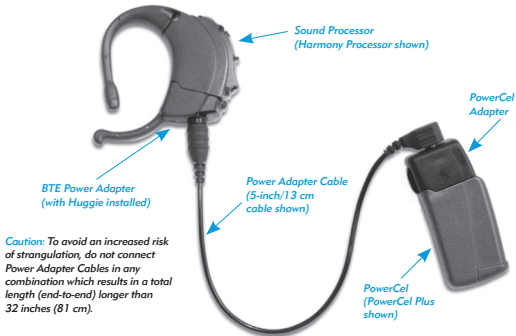
Figure 1.

Using the PowerCel Adapter to Power the Auria or Harmony Processor

To Connect the Power Adapter Cable to the PowerCel Adapter

- Insert the metal plug of the Cable into the socket on the PowerCel Adapter (Figure 2).

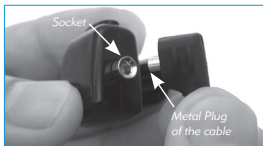
Figure 2.

To Connect the Power Adapter Cable to the BTE Power Adapter

- Connect the socket end of the Power Adapter Cable to the plug on the underside of the BTE Power Adapter (Figures 3 and 4).

Figure 3. BTE Power Adapter with Huggie installed.

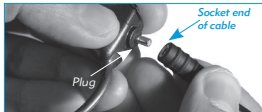
Figure 4.

To Connect the BTE Power Adapter to the Auria or Harmony Processor

- Guide the BTE Power Adapter onto the tracks on the bottom of the Processor (Figure 5).

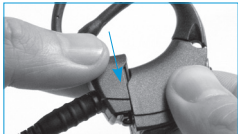
Figure 5.

- Once the BTE Power Adapter is fully engaged on the Processor, push the latch (Figure 3) up until it snaps to lock.
- To remove: If necessary, remove the headpiece cable. Pull the latch down and slide the Adapter off the Processor.

To Connect the PowerCel to the PowerCel Adapter

- Push the latch on the PowerCel Adapter towards the cable connector (Figure 6).

Figure 6.

- Guide the PowerCel onto the tracks on the bottom of the PowerCel Adapter (Figure 7). Once the PowerCel is fully engaged on the Adapter, slide the latch toward the PowerCel to lock.

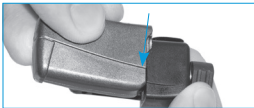
Figure 7.

- Always remove the PowerCel when reconfiguring the external hardware. Once all the other parts are connected properly, reconnect the PowerCel.

Note: Please remove the PowerCel from the adapter if the adapter is not going to be used for a long time.

Preparing your BTE Power Adapter for use with the Huggie or Filler Plug

The BTE Power Adapter is shipped with the Huggie (Figure 8) installed (Figure 3), and includes a Filler Plug (Figure 9) for optional use.

Figure 8 Huggie

Figure 9 Filler Plug

To Remove the Huggie

- Loosen the screw using a small flat-blade screwdriver such as an eyeglass screwdriver (Figure 10).
- Remove the screw and set it aside.
- Gently slide the Huggie out.

Figure 10

To Attach the Filler Plug

- Gently push the Filler Plug into the open slot at the bottom of the BTE Power Adapter.
- Place the screw in the top of the Filler Plug.
- Using a small flat-blade screwdriver, gently tighten the screw.

To Remove the Filler Plug

- Loosen the screw.
- Gently push on the screw until the Filler Plug is loosened and easy to grasp.
- Gently pull the Filler Plug out.
- Remove the screw.

Installing the Cable Clip to the Power Adapter Cable

The Power Adapter Cable is shipped with a Cable Clip. To install the Cable Clip, slide the cable into one of the slots of the smaller clip (Figure 11).

Figure 11

Wearing the PowerCel Adapter with a Safety Pin

The PowerCel Adapter can be worn using a safety pin through the hole at the back of the clip (Figure 12). The pin can then be attached to an available piece of clothing.

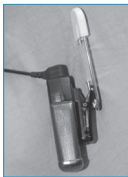
Figure 12

Maintenance, Protection and Cleaning

Avoid the following:

- Operating the PowerCel Adapter in extreme temperatures (below 0°F/32°C or above 45°F/115°C).
- Storing the PowerCel Adapter in extreme temperatures (below -20°F/-4°C or above 55°F/131°C).

The PowerCel Adapter meets the temperature safety requirements of IEC 60601-1:2005 in ambient temperatures up to 33°F/91°C.

- Dropping the products on a hard surface.
- Pulling on the cable or bending the cable sharply.
- Pushing on the connectors in a direction other than the direction required to connect or remove the connector.

Cleaning

- Always disconnect the PowerCel prior to cleaning.
- If necessary, the PowerCel Adapter and the BTE Power Adapter can be cleaned with a slightly dampened cloth or tissue.
- Do not use soap or solvent.

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©2023 Advanced Bionics LLC. All Rights Reserved.
Advanced Bionics LLC
Advanced Bionics GmbH