DRG:HYBRID•XL®

Instructions for Use

Pancreatic Elastase

IVD

REF

HYE-6123

40 tests

DRG Instruments GmbH, Germany
Frauenbergstraße 18, D-35039 Marburg
Phone: +49 (0)6421-1700 0, Fax: +49 (0)6421-1700 50
Website: www.drg-diagnostics.de
E-mail: drg@drg-diagnostics.de

Distributed by:

DRG International, Inc., USA
841 Mountain Ave., Springfield, NJ 07081
Phone: (973) 564-7555, Fax: (973) 564-7556
Website: www.drg-international.com
E-mail: corp@drg-international.com

ASSAY PROTOCOL BARCODE (APB)

**(Version 2.61 Software or later / ab Softwareversion 2.61 oder höher / Versione Software 2.61 o superiore /
 Versión de Software 2.61 o posterior / Version de logiciel 2.61 et supérieure /
 Wersja oprogramowania 2.61 lub późniejsza)**

PELA - HYE-6123 - 02.61.01

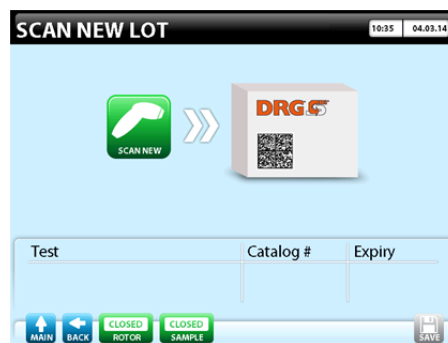
HYE-6123 – v2.61.1

The barcode must be used to install the assay protocol into the DRG:HYBRiD-XL software via the SCAN NEW LOT page.
 Der Barcode muss in dem Menü „NEUES LOT SCANNEN“ eingelesen werden, um das Protokoll in der DRG:HYBRiD-XL-Software zu installieren.

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Please refer to section **3: Routine Procedures: "Installing a new assay product"** of the User Manual v2.60 or later.

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Contents / Inhaltsverzeichnis / Contenuti / Contenido / Contenu / Spis treści

1	INTRODUCTION	4
2	PRINCIPLE OF THE TEST	4
3	WARNINGS AND PRECAUTIONS	4
4	REAGENTS	4
5	SPECIMEN COLLECTION AND PREPARATION	5
6	ASSAY PROCEDURE	5
7	QUALITY CONTROL	6
8	EXPECTED NORMAL VALUES	6
9	LIMITATIONS OF USE	6
10	PERFORMANCE CHARACTERISTICS	6
11	METHOD COMPARISON	7
12	LEGAL ASPECTS	7

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1 INTRODUCTION

1.1 Intended Use

The **DRG:HYBRID-XL Pancreatic Elastase** is an enzyme immunoassay for the quantitative *in vitro diagnostic* measurement of **Pancreatic Elastase** in human **stool samples**.

Only for use with the DRG:HYBRID-XL Analyzer.

1.2 Summary and Explanation

The pancreatic elastase (Chymotrypsin-like elastase family member 3 (CELA 3)) is a proteolytic glycoprotein of the serine protease family with a molecular weight of approx. 28 kDa (1). It is secreted as a pro-enzyme by the acinar cells of the pancreas via the Papilla Vateri into the duodenum (2,7). After the activation of pancreatic elastase, it cleaves peptides after neutral amino acids such as trypsin (5). The concentration of the pancreatic elastase is six-fold higher in human faeces than in the pancreatic juice. Pancreas elastase is bound to bile salts and is not degraded during the intestinal passage (1).

In humans, two groups of elastases can be discriminated (CELA 2A and B and CELA 3A and B). Analysis of CELA 3 in stool samples reflects the secretory capacity of the pancreas (exocrine function of the pancreas) (8).

Reduced pancreatic elastase concentration is an indicator of chronic pancreatitis or pancreas insufficiency. Pancreatic insufficiency is the inability of the exocrine pancreas to produce and/or transport enough digestive enzymes to break down food in the intestine and facilitate absorption of nutrients. It typically occurs as a result of progressive pancreatic damage that may be caused by recurrent acute pancreatitis or by chronic pancreatitis due to a variety of conditions (e.g. cystic fibrosis, pancreatitis or diabetes mellitus, gallstones, alcohol abuse or pancreas carcinoma) (3). Pancreatic insufficiency causes maldigestion and malabsorption which are followed by vitamin, mineral and trace element deficiency. Calcium deficiency increases the risk for osteoporosis and osteomalacia.

2 PRINCIPLE OF THE TEST

The **DRG:HYBRID-XL Pancreatic Elastase** Kit is a solid phase enzyme-linked immunosorbent assay (ELISA) based on the **sandwich principle**.

The antibody coated wells (ACW) of the reagent cartridges are coated with a monoclonal (mouse) antibody directed towards a unique antigenic site of the pancreatic elastase molecule. An aliquot of patient sample containing endogenous pancreatic elastase is incubated in the coated well with enzyme conjugate, which is an anti-pancreatic elastase antibody conjugated with horseradish peroxidase. After incubation the unbound conjugate is washed off.

The amount of bound peroxidase conjugate is proportional to the concentration of pancreatic elastase in the sample.

Having added the substrate solution, the intensity of colour developed is proportional to the concentration of pancreatic elastase in the patient sample.

3 WARNINGS AND PRECAUTIONS

1. This kit is for *in vitro* diagnostic use only.
For professional use only.
2. This kit can only be used in combination with the DRG:HYBRID-XL Analyzer
3. Before starting the assay, read the instructions completely and carefully. Use the valid version of the package insert provided with the kit. Be sure that everything is clear and understood.
4. **Do not remove, exchange, discard or damage any of the barcode labels provided with each kit and its components. All barcodes build an integral system for the kit lot.**
5. Respect the general safety measures for use of laboratory reagents.
6. All reagents of this test kit which contain human serum or plasma have been tested and confirmed negative for HIV I/II, HBsAg and HCV by FDA approved procedures. All reagents, however, should be treated as potential biohazards in use and for disposal.
7. Never pipet by mouth and avoid contact of reagents and specimens with skin and mucous membranes.
8. Do not smoke, eat, drink or apply cosmetics in areas where specimens or kit reagents are handled.
9. Wear appropriate disposable gloves when handling specimens and reagents. Microbial contamination of reagents or specimens may cause false results.

10. Handling should be done in accordance with the procedures defined by an appropriate national biohazard safety guideline or regulation.
11. Do not use reagents beyond expiry date as shown on the kit labels.
12. Unused reagent cartridges must be stored at 2 °C to 8 °C in the sealed foil pouch with desiccant provided.
13. Optimal test results are only obtained when using calibrated pipettes.
14. Do not mix or use components from kits with different lot numbers. It is advised not to interchange reagent cartridges of different kits even of the same lot. The kits may have been shipped or stored under different conditions and the binding characteristics of the wells in the reagent cartridges may differ slightly.
15. Some reagents contain Proclin 300, BND and/or MIT as preservatives. In case of contact with eyes or skin, flush immediately with water.
16. TMB substrate has an irritant effect on skin and mucosa. In case of possible contact, wash eyes with an abundant amount of water and skin with soap and plenty of water. Wash contaminated objects before reusing them. If inhaled, take the person to open air.
17. Chemicals and prepared or used reagents have to be treated as hazardous waste according to the national biohazard safety guidelines or regulations.
18. For information on hazardous substances included in the kit please refer to Safety Data Sheets.
For professional users the Safety Data Sheet for this product is available upon request directly from DRG.

4 REAGENTS

4.1 Reagents provided

4.1.1 Reagent Cartridges

40 pieces containing the following:

- **Antibody Coated Well (ACW)**
coated with anti-pancreatic elastase antibody (monoclonal).
- **Enzyme Conjugate**, 170 µL,
Anti-pancreatic elastase antibody conjugated with horseradish peroxidase;
Contains non-mercury preservative.
- **Assay Diluent**, 310 µL;
- **Substrate Solution**, 270 µL
Tetramethylbenzidine (TMB).

4.1.2 Re-Calibrator 1 & 2

2 vials, 0.5 mL each, ready to use;

For re-calibration of the quantitative **DRG:HYBRID-XL Pancreatic Elastase** test.

Concentrations are lot-specific.

Re-Calibrators are standardised against the following reference material: INSTAND ring trial No.130 Fecal Diagnostics 01

Contain non-mercury preservative.

4.1.3 Control 1 & 2

2 vials, 0.5 mL each, ready to use;

For control values and ranges please refer to the bar code on vial label or to the QC-Datasheet.

Contain non-mercury preservative.

4.2 Materials required but not provided

- General needed laboratory equipment
- Ultra-pure water
DRG recommends to use Clinical Laboratory Reagent Water (CLRW) according to CLSI guideline 3C-A4 with the following specifications:
Resistivity at 25 °C [MΩ·cm]: > 10
Conductivity at 25 °C [µS·cm⁻¹]: < 0.1
Total Organic Carbon/p.p.b.[µg/L]: < 50
Colloids [µg/mL]: <0.05
- [REF] HYB-5252 DRG:HYBRID-XL Analyzer
- [REF] HYI-5392: *System Solution 5L*, 5000 mL;
(Instrument Feed Water according to CLSI guideline 3C-A4 with the following specification can also be used:
Resistivity at 25 °C [MΩ·cm]: > 1
Conductivity at 25 °C [µS·cm⁻¹]: < 1
Total Organic Carbon/p.p.b.[µg/L]: < 200
Colloids [µg/mL]: <0.1)
- [REF] HYI-5394: *Wash Buffer*, 40x concentrate, 25 mL

- [REF] HYI-5395: *Needle Cleaning Solution*, 30 mL. Cleaning solution for the pipetting needle (daily and weekly maintenance, see also user manual)
- [REF] HYI-5387: *Cuvettes*, (2 x 360 pieces)

For use of the *Secondary Sample Holder* for secondary tubes the following tubes are required:

- [REF] HYI-5391: *Sample Tubes (Secondary)*, 2500 pcs.

4.3 Storage Conditions

All kit components should be stored at 2 °C to 8 °C to ensure product performance until the defined expiry date.

When stored at 2 °C to 8 °C, **unopened kits** will retain reactivity until expiration date. Do not use reagents beyond this date.

- Cartridges (stored at 2 °C to 8 °C) in the supplied and unopened zip/foil bags will retain reactivity until expiration date.
- Unopened Re-Calibrators and Controls (stored at 2 °C to 8 °C) will retain reactivity until expiration date.

Opened reagents and the reagent cartridges must be stored at 2 °C to 8 °C.

Once the plastic bag has been opened, care should be taken to tightly close it again along with the supplied desiccant bag.

Immediately after end of each run the Re-Calibrator and Control vials have to be removed from the instrument, tightly capped and stored at 2 °C to 8 °C.

- Unused cartridges in opened zip/foil bags (stored at 2 °C to 8 °C) will retain reactivity until expiration date, if stored as described above.
- Pierced or open cartridges must be disposed of immediately.
- Opened Re-Calibrators and Controls (stored at 2 °C to 8 °C) will retain reactivity for 8 weeks.

4.3.1 On-board Stability

For Re-Calibrators and Controls the on-board stability has been evaluated under controlled laboratory conditions at room temperature (20 °C to 25 °C).

Due to the differences in laboratory environmental conditions and reagent volumes, the on-board stability may deviate from the declared value.

On-board stability:	24 h
---------------------	------

4.4 Reagent Preparation

Bring all reagents, such as controls and re-calibrators, to room temperature (20 °C to 25 °C) prior to use. Reagent Cartridges can be used directly after storage in the refrigerator.

Wash Buffer (not included in the kit)

For Wash Buffer (1x) dilute 25 mL of *Wash Buffer* (40x) with 975 mL ultra-pure water to a final volume of 1000 mL.

The diluted Wash Buffer (1x) is stable for 2 weeks at room temperature.

4.5 Disposal of the Kit

The disposal of the kit and all used materials/reagents must be performed according to the national regulations. Special information for this product is given in the Safety Data Sheet.

4.6 Damaged Test Kits

In case of any damage to the test kit or components, DRG must be informed in writing, at the latest one week after receiving the kit. Damaged single components should not be used for a test run. They have to be stored until a final solution has been found. After this, they should be disposed of according to the official regulations.

5 SPECIMEN COLLECTION AND PREPARATION

Stool extracts from raw stool can be used in this assay.

It is recommended to note the consistency of very watery stools. In case of pathological result (< 200 µg/g) a formed stool sample should be requested.

Attention: Do not use external controls stabilized with azide.

5.1 Specimen Collection

We **strongly** recommend the use of the *Fecal Extraction System 2.6* (FES; [REF]: HYI-5989).

Only the *Fecal Extraction System 2.6* (FES; [REF]: HYI-5989) can be used for stool sample preparation in the *DRG:HYBRiD-XL Pancreatic Elastase* test.

No other stool sample extraction system has been tested for the *DRG:HYBRiD-XL Pancreatic Elastase* test (HYE-6123).

For sample preparation please follow the Instructions for Use included in the package of HYI-5989.

After finishing the raw stool sample preparation in the FES tubes, the stool extracts are ready for use in the *DRG:HYBRiD-XL Pancreatic Elastase* test.

Place the opened extraction tube into the adapter for the primary arc.

A minimum of 68 µL of stool extract is needed for one determination. This includes 8 µL sample and 60 µL dead volume.

5.2 Specimen Storage

Raw Stool

Pancreatic Elastase in stool is described to be stable for 2 days at 2 °C to 8 °C.

Specimens stored for a longer time (up to 12 months) should be frozen only once at -20 °C prior to the assay.

Allow frozen samples to thaw slowly, preferably at 2 °C - 8 °C, and warm the samples to room temperature before analysis. Avoid repeated freezing and thawing of the sample.

Stool Extract

Extracts should be capped and may be stored for up to 7 days at 2 °C to 8 °C prior to performing the assay.

Stool extracts stored for a longer time (up to 12 months) should be frozen only once at -20 °C prior to the assay. Thawed samples should be inverted several times prior to testing.

6 ASSAY PROCEDURE

6.1 General Remarks

- All reagents, such as controls and re-calibrators and specimens, must be allowed to come to room temperature (20 °C to 25 °C) before use.
- All reagents and samples must be mixed without foaming. Reagent Cartridges can be used directly after storage in the refrigerator.
- Samples, controls and re-calibrators should be measured within 2 hours in order to avoid possible evaporation effects.
- The *Secondary Sample Holder* (HYI-5437) for secondary tubes has the capacity for a maximum of 20 samples including controls and re-calibrators. They all have to be pipetted into the secondary tubes, and the respective barcodes of control/re-calibrator vials and, if available, the sample barcodes have to be read with the external barcode scanner.

6.2 Test Procedure

- **The total assay time for *DRG:HYBRiD-XL Pancreatic Elastase* is 90 minutes.**
- To ensure proper operation of the test, the instructions in the user manual for the *DRG:HYBRiD-XL* should strictly be followed.
- All test specific information required for the correct operation is included in the respective barcodes of the reagents.
Take care not to damage these bar codes!
- **It is recommended to tap the bottom of the Cartridge Segments containing the reagent cartridges once on the bench before placing them on the rotor. This is to avoid foam and adhering of the liquid on the sealing of the reagent cartridge.**
- Place reagent cartridges on the rotor of the unit. The heating to 37 °C incubation temperature is performed automatically in the unit.

6.3 Calibration

Traceability:

This method was standardized against: INSTAND ring trial No.130 Fecal Diagnostics 01.

Each DRG:HYBRiD-XL reagent contains a barcode with the specific information for recalibration of the reagent lot. The Master Curve is printed as a 2-D barcode on the outer label of the kit package and on the QC-Datasheet and has to be scanned with the external barcode scanner prior to the first use of the respective kit lot.

Recalibration is recommended:

- if a new kit lot is used. Each new lot should be verified by running the kit internal re-calibrators and controls before routine use.
- if one or both assay controls are found outside the specified range.
- after 4 weeks of use of the same reagent kit on the unit.

6.4 Calculation of Results

The analyte concentrations are calculated automatically by the DRG:HYBRiD-XL's system software.

7 QUALITY CONTROL

It is recommended to use control samples according to state and federal regulations. The use of control samples is advised to assure the day to day validity of results.

It is also recommended to participate in national or international Quality Assessment programs in order to ensure the accuracy of the results.

Apply appropriate statistical methods for analysing control values and trends. If the results of the assay do not agree with the established acceptable ranges of control materials, patient results should be considered invalid. In this case, please check the following: expiration dates and storage conditions of reagents, operational reliability of the analyser. In addition, it is indicated to perform a Recalibration.

In case of further questions please contact your local distributor or DRG directly.

7.1 Internal Controls

For Quality Control it is necessary to use the two internal controls provided with each kit.

Acceptance ranges for both internal controls (*Control 1 & 2*) were established by the manufacturer and are summarized in the QC-Datasheet added to the kit. Note that the expected values and acceptance ranges stated in the QC-Datasheet always refer to the current kit lot.

Internal controls should be run in single determination:

- on a routine basis (e.g. once per 24 h)
- if re-calibration is required (if one or both internal controls are out of range)
- if a new kit lot is used (in order to avoid any negative impact on the kit performance by improper transport or to detect improper storage during transport).

7.2 External Controls

Use controls at both normal and pathological levels.

The control intervals and control ranges for external controls should be adapted to the individual requirements of each laboratory. All results must be within the defined limits.

Each laboratory should establish corrective measures to be taken if values of external controls are not found in the acceptance range.

8 EXPECTED NORMAL VALUES

It is strongly recommended that each laboratory should determine its own normal and pathological values.

In a study conducted with apparently healthy adults, using the **DRG:HYBRiD-XL Pancreatic Elastase** the following data were observed:

Population	Healthy adults
n	82
Range (min. - max.) (µg/g)	200.0 - 581.0
Mean (µg/g)	344.1
2.5 th - 97.5 th Percentile (µg/g)	200.6 – 542.0
Median (µg/g)	324.5

A review of the literature suggests the following ranges for the classification of pancreatic insufficiency:

Status	Pancreatic Elastase (µg/g stool)
Normal	> 200
Moderate to weak pancreas insufficiency	100 - 200
Severe pancreas insufficiency	< 100

The results alone should not be the only reason for any therapeutic consequences. The results should be correlated with other clinical observations and diagnostic tests.

9 LIMITATIONS OF USE

Reliable and reproducible results will be obtained, when the assay procedure is performed with a complete understanding of the instructions for use and with adherence to good laboratory practice. Any improper handling of samples or modification of this test might influence the results.

10 PERFORMANCE CHARACTERISTICS

10.1 Assay Dynamic Range

The dynamic range of the assay is defined by the limit of detection and the maximum value of the Master Curve.

Values found below the measuring range are indicated as "< 10.52 µg/g".

Values found above the measuring range are indicated as "> 540.0 µg/g".

The measuring range of the assay is between 10.52 µg/g – 540.0 µg/g.

10.2 Specificity of Antibodies (Cross-Reactivity)

The following substances were tested for cross-reactivity of the assay:

Substances	Added conc. (µg/mL)	Cross-reactivity (%)
Amylase	50 - 5000	0.0
Chymotrypsin	50 - 5000	0.0
Calprotectin	93 - 1500	0.0
Lipase	50 - 5000	0.0
Anti-1-trypsin	50 - 5000	0.0
Pancreatin (pig)	50 - 5000	0.0

10.3 Sensitivity

The sensitivity study was designed according to CLSI guideline EP17-A2.

The Limit of Blank (LoB) is 1.330 µg/g.

The Limit of Detection (LoD) is 10.52 µg/g.

The Limit of Quantification (LoQ) is 28.94 µg/g.

10.4 Precision Performance

The precision study was designed on the basis of to CLSI guideline EP5-A2.

10.4.1 Intra-Device Precision

The Intra-Device precision was determined with 4 patient samples covering the measuring range in 5 independent runs on 5 days with 2 different devices in 5 replicates per run. CV was calculated as mean CV of 10 runs.

The within-run variability is shown below:

Sample	n	Mean (µg/g)	CV (%)
1	10	102.6	2.1
2	10	192.1	2.8
3	10	213.8	2.8
4	10	416.9	5.4

10.4.2 Inter-Device Precision

The Inter-Device precision was determined with 4 patient samples covering the measuring range in 5 independent runs on 5 days with 2 different devices with 5 determinations per run. CV was calculated from 50 determinations.

The total precision is shown below:

Sample	n	Mean (µg/g)	CV (%)
1	50	102.6	3.4
2	50	192.1	4.4
3	50	213.8	5.3
4	50	416.9	6.2

10.5 Recovery

Recovery was determined by adding four increasing concentrations of the analyte to four different patient samples containing different amounts of endogenous analyte. Each sample (non-spiked and spiked) was assayed and analyte concentrations of the samples were calculated from the Master Curve. The percentage recoveries were determined by comparing expected and measured values of the samples.

Sample	1	2	3	4
Concentration (µg/g)	227.5	352.5	356.5	459.0
Average Recovery (%)	110.8	94.8	92.3	92.3
Range of Recovery (%)	from 88.3	93.2	91.5	90.2
	to 119.9	97.2	93.2	96.6

10.6 Interfering Substances

Haemoglobin (up to 2 mg/mL) and Triglyceride (up to 1.88 mg/mL) have no influence on the assay results.

Until today no substances (drugs) are known to us, which have an influence to the measurement of *Pancreatic Elastase* in a sample.

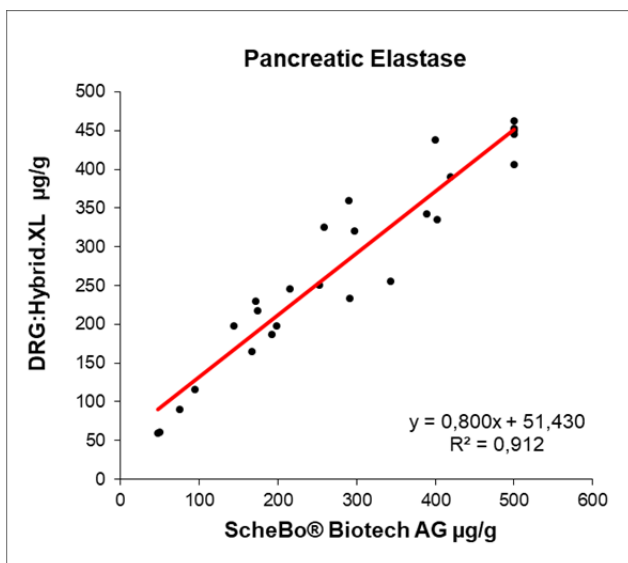
10.7 High-Dose-Hook Effect

Hook effect was not observed in this test up to a concentration of 540.0 µg/g of pancreatic elastase.

11 METHOD COMPARISON

A comparison of **DRG:HYBRiD-XL Pancreatic Elastase Test** HYE-6123 (y) and **ScheBo® Pancreatic Elastase 1™ Stool Test (x)** using clinical samples gave the following correlation:

$$\begin{aligned} n &= 26 \\ r &= 0.955 \\ y &= 0.800x + 51.430 \end{aligned}$$

12 LEGAL ASPECTS

12.1 Reliability of Results

The test must be performed exactly as per the manufacturer's instructions for use. Moreover, the user must strictly adhere to the rules of GLP (Good Laboratory Practice) or other applicable national standards and/or laws. This is especially relevant for the use of control reagents. It is important to always include, within the test procedure, a sufficient number of controls for validating the accuracy and precision of the test.

The test results are valid only if all controls are within the specified ranges and if all other test parameters are also within the given assay specifications. In case of any doubt or concern please contact DRG.

12.2 Therapeutic Consequences

Therapeutic consequences should never be based on laboratory results alone even if all test results are in agreement with the items as stated under point 12.1. Any laboratory result is only a part of the total clinical picture of a patient.

Only in cases where the laboratory results are in acceptable agreement with the overall clinical picture of the patient should therapeutic consequences be derived.

The test result itself should never be the sole determinant for deriving any therapeutic consequences.

12.3 Liability

Any modification of the test kit and/or exchange or mixture of any components of different kit lots could negatively affect the intended results and validity of the overall test. Such modification and/or exchanges invalidate any claim for replacement.

Claims submitted due to customer misinterpretation of laboratory results subject to point 12.2 are also invalid. Regardless, in the event of any claim, the manufacturer's liability is not to exceed the value of the test kit. Any damage caused to the test kit during transportation is not subject to the liability of the manufacturer.

For further information please refer to the User Manual of the DRG:HYBRiD-XL, analyser-specific application sheets, product information and package inserts of all necessary components.

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1. Whitcomb, DC, Lowe, ME. Human pancreatic digestive enzymes. *Digestive diseases and sciences* 2007, 52, 1–17.
2. Hammel, I, Meilijson, I. The econobiology of pancreatic acinar cells granule inventory and the stealthy nano-machine behind it. *Acta Histochem* 2016, 118, 194–202.
3. Altay M. Which factors determine exocrine pancreatic dysfunction in diabetes mellitus? *World J Gastroenterol* 2019; 14; 2699-2705.
4. Capurso, G, Traini, M, Piciocchi, M, Signoretti, M, Arcidiacono, PG. Exocrine pancreatic insufficiency: Prevalence, diagnosis, and management. *Clin. Exp. Gastroenterol.* 2019; 12:129–139.
5. Leeds, JS, Oppong, K, Sanders, SD. The role of fecal elastase-1 in detecting exocrine pancreatic disease. In: *Nature reviews. Gastroenterology & hepatology.* Band 8/7, Juli 2011, 405–415.
6. Sikkens, ECM. Pancreatic enzyme replacement therapy in chronic pancreatitis; *Best Practice & Research Clinical Gastroenterology* 24 2010; 337-347.
7. Hardt, PD et al. Pathological Pancreatic Exocrine Function and Duct Morphology in Patients with Cholelithiasis; *Digestive Diseases and Sciences*, Vol. 46, No. 3, pp. 536–539
8. (8) Szepessy, E, Sahin-Tóth, M. Inactivity of Recombinant ELA2B Provides a New Example of Evolutionary Elastase Silencing in Humans, *Pancreatology* 2006; 6:117–122.

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