

Step 1 Entry criteria

Does the wound fit into one of the categories below?

- Wound is not clinically infected[†]
- None of the standard contraindications for NPWT apply

Yes Continue to Step 2

No Not PICO appropriate

Step 2 Exudate levels

Does the wound have low to moderate levels of exudate?



Yes Continue to Step 3

No Not PICO appropriate

Step 3 Wound size

Does the wound fit under 1 of the 10 PICO Dressings?

Yes Continue to Step 4

No Not PICO appropriate

Recommendation: Choosing a dressing size that is slightly larger than the wound allows the benefits of negative pressure to extend to the periwound area. When used over a moderately exuding wound, the size of the wound should generally be no more than 25% of the dressing pad area.

Ordering information

		PICO 7 Dressing Size 2.0cm x 2.0cm	PICO 14 Dressing Size 2.0cm x 2.0cm	PICO 14 Dressing Size 2.0cm x 2.0cm
Ordering	Ordering Item	2.0 x 2.0cm (4in)	2.0 x 2.0cm (4in)	2.0 x 2.0cm (4in)
	PICO Dressing	WAND000	WAND000	WAND000
	PICO Dressing	WAND001	WAND001	WAND001
	PICO Dressing	WAND002	WAND002	WAND002
	PICO Dressing	WAND003	WAND003	WAND003
	PICO Dressing	WAND004	WAND004	WAND004
	PICO Dressing	WAND005	WAND005	WAND005
	PICO Dressing	WAND006	WAND006	WAND006
	PICO Dressing	WAND007	WAND007	WAND007
	PICO Dressing	WAND008	WAND008	WAND008
	PICO Dressing	WAND009	WAND009	WAND009
	PICO Dressing	WAND010	WAND010	WAND010
Contraindications	Product Code			
	PICO Dressing	WAND001	WAND001	WAND001
	PICO Dressing	WAND002	WAND002	WAND002
	PICO Dressing	WAND003	WAND003	WAND003
	PICO Dressing	WAND004	WAND004	WAND004
	PICO Dressing	WAND005	WAND005	WAND005
	PICO Dressing	WAND006	WAND006	WAND006
	PICO Dressing	WAND007	WAND007	WAND007
	PICO Dressing	WAND008	WAND008	WAND008
	PICO Dressing	WAND009	WAND009	WAND009
	PICO Dressing	WAND010	WAND010	WAND010

Note: Dressing size includes a 1 inch silicone border around the entirety of the perimeter

Step 4 Wound depth

Note: NPWT requires direct contact with the wound bed. Prior to application, assess the wound to determine depth and if a filler would be required

Does the PICO system conform to the wound bed?


Yes Begin application*

No Continue to Step 5, prior to application*

Wound bed depth <0.5cm. No wound filler required



Wounds greater than 0.5cm (1/4in) in depth are likely to require a foam or gauze negative pressure woundtherapy filler to ensure adequate treatment of all the wound surfaces. Wounds greater than 2cm (1/2in) in depth must be treated with the use of a filler along with the PICO 7 system to ensure adequate contact with the wound.

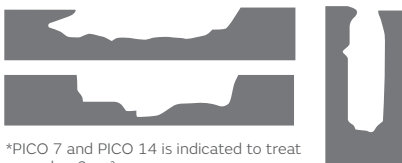

PICO 7 and PICO 14 is indicated to treat wounds > 2cm.

Step 5 Fillers



Antimicrobial gauze

Note: Gauze should loosely fill to the surface of the wound. Avoid over packing



Foam wound dressing 10cm x 12.5cm

Note: Please refer to the IFU for information regarding possible tissue ingrowth when using foam filler



ACTICOAT[®] FLEX 3

Note: ACTICOAT Flex 3 and ACTICOAT Flex 7 are only approved for use with NPWT for up to 3 days

Wear instructions:

- At the healthcare professional's discretion, a PICO dressing may be left in place for up to 7 days
- When a filler is used, the filler and the PICO Dressing should be changed 2 to 3 times a week, according to the local clinical protocol and manufacturer's instructions

Some examples of appropriate wounds



Diabetic foot ulcer

- Low to moderate exudate



Venous leg ulcer

- Low to moderate exudate



Traumatic wound

- Low to moderate exudate



Pressure ulcer

- Low to moderate exudate



Surgical dehiscence

- Low to moderate exudate
- Filler recommended



Amputation site

- Low to moderate exudate
- Filler recommended
- Conjunctive therapy with offloading

Note: Please refer to the IFU for a full list of Indications and Contraindications

Recent clinical studies have found that...

61%

of wounds healed or were progressing to healing at 26 weeks¹

13.4%

reduction in weekly average of wound areas during PICO treatment, compared to pre-PICO rate (P=0.006)¹

81%

of patients reported that they were pleased with treatment²

86%

of clinicians rated PICO as easy to apply²

Ordering information

Dressing	Dressing size	PICO 7 device and 2 dressings	PICO 14 device and 2 dressings	Multi pack with 5 dressing packs
	Multisite small 15cm x 20cm	2 x dressing kit*	2 x dressing kit**	Fluid Management Packs***
	Multisite large 20cm x 25cm	66802001	66802041	66802021
	10cm x 20cm	66802002	66802042	66802022
	10cm x 30cm	66802003	n/a	66802023
	10cm x 40cm	66802004	n/a	66802024
	15cm x 15cm	66802005	66802045	66802025
	15cm x 20cm	66802006	66802046	66802026
	15cm x 30cm	66802007	66802047	66802027
	20cm x 20cm	66802008	66802048	66802028
	25cm x 25cm	66802009	66802049	66802029
Consumables		Product code		
	Foam dressing filler	10cm x 12.5cm	66801021	
	RENASYS® Adhesive Gel Patch	10cm x 7cm	66801082	
	ACTICOAT® Flex 7	2.5cm x 60cm	66800545	
	ACTICOAT Flex 3	10cm x 10cm	66800399	ACTICOAT Flex 3 and ACTICOAT Flex 7 are only approved for use with NPWT for up to 3 days

For detailed product information, including indications for use, contraindications, effects, precautions and warnings, please consult the product's Instructions for Use (IFU) prior to use.

References: 1. Dowsett C et al. Use of PICO to improve clinical and economic outcomes in hard-to-heal wounds. *Wounds Int* 2017;8(2):53-58. A prospective co-hort study of 52 wounds. 2.Hurd, T., Trueman, P., & Rossington, A. Use of portable, single use negative pressure wound therapy device in home care patients with low to moderately exuding wounds: a case series. *Ostomy Wound Management*. Volume 60. Issue 3. March 2014. 3.Smith & Nephew Clinical, Scientific and Medical Affairs Evidence Outcomes Report. The Review Of Evidence Supporting The Use Of PICO In Wounds ≥2cm In Depth Report Ref: EO.AWM.PCS230.001.v2